

RxCrush Pill Pouches

PP06, PP08 Sealed ENFit® Pill Pouches



- PP06 - 7.6cm x 12.5cm, 50ml
- PP08 - 5.1cm x 7.6cm, 30ml
Ideal for pediatric patients
- Locking ENFit® (ISO 80369-3 compliant) connects directly to syringe
- Zip seal contains contents
- Offset flaps for easier pill insertion
- Puncture resistant

PP10, PP11 Sealed Tear-off Pill Pouches



- PP10 - 5.1cm x 12.5cm, 30ml
Ideal for pediatric patients
- PP11 - 7.6cm x 17.8cm, 50ml
- Large offset flaps for easier pill insertion
- Contains medication while crushing
- Tear-off top easily empties contents
- Puncture resistant

PP14 Open Pill Pouch



- PP14 - 5.1cm x 12.5cm
- Open pouch for quick pill insertion
- Puncture resistant

CONTACT US

Telephone

+01.877.343.2783


E-Mail

sales@rxcrush.com

Website

www.rxcrush.com

FDA Registered, 

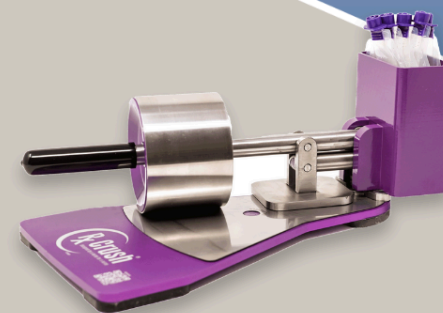

Pill Crushing System

**Pulverize
Contain
Deliver**

How to RxCrush



PULVERIZE pills in the RxCrush pill pouch by first crushing and then rolling. Pill dust is then ready to be dissolved in liquid or added to food.



CONTAIN pills in the durable sealed RxCrush pill pouch during pulverization.



Add liquid from a syringe to the pulverized medication in the pill pouch. Shake the mixture to dissolve, and **DELIVER** directly to a feeding tube via the syringe.

instructions



Pill Pouch Caps

Use pill pouch **CAPs** to seal pouches for transport or storage. Push-on or screw-on options available.



Haven't transitioned to ENFit yet?

Use Luer, Oral and Catheter/Bolus **ADAPTERS** to connect RxCrush ENFit® pouches to your existing syringes and feeding sets.



RXBR RxCrush Brochure Rev E, 10/16/2025



Manufactured by Resolve Designs
PO Box 572, Springville, UT 84663, US



CMC Medical Devices & Drugs S.L.
C/ Horacio Lengo No 18, CP29006, Malaga Spain
info@cmcmedicaldevices.com

RxCrush is a registered trademark of Resolve Designs
US Patents 9,180,071 & 11,096,868