



Resolve Designs

PC16INENCLS RxCrush Pill Crusher II Cleaning Steps

RevA

Instructions for Cleaning RxCrush® Pill Crusher II (PC16)

Thank you for choosing the RxCrush® Pill Crusher II. Proper cleaning is essential for maintaining your device and ensuring patient safety. The RxCrush® Pill Crusher II is designed for easy cleaning with water and hospital-grade disinfectant cleaners.

We recommend using RxCrush® Pill Pouches with ENFit or Sealed Tear-off pill pouches (all which are single-patient use pill pouches) to minimize the need for frequent cleaning. However, if there is a spill or visible residue, follow the steps below.

Routine Cleaning (When there is visible medication residue)

1. **Remove the pouch caddy.** The pouch caddy should be removed before cleaning the main unit to ensure all surfaces are accessible.
2. **Wipe down the crusher.** Use an approved disinfectant wipe to clean all exterior surfaces of the crusher, including the handle, armature, roller, and crush plate.

Deep Cleaning (When there is visible medication residue)

1. **Remove the pouch caddy.** As with routine cleaning, remove the pouch caddy first to access the internal components for a thorough clean.
2. **Apply a cleaning agent.** Spray one of the approved cleaning agents listed below directly onto the affected areas.
 - **Approved Chemicals:**
 - Chlorine-based wipes
 - Alcohol-based wipes
 - Hydrogen peroxide-based wipes
 - Examples: PDI - Super Sani Cloth and Clorox Healthcare Hydrogen Peroxide Cleaner Disinfectant Wipes. (See [INENCL RxCrush Cleaning Guide RevA](#) for recommended cleaning agents)
3. **Wipe and dry.** Thoroughly wipe down the interior and exterior of the crusher. Allow all surfaces to air dry completely before the next use.

Important Note: The RxCrush® pill pouches are for single-patient use only and should not be cleaned or reused. Always dispose of used pouches properly to prevent cross-contamination.